



Digital Thermometer : HK-902

- Flexible tip/with steel tip
- Response: 60 seconds
- Transparent PP case
- Measuring Range: 32.0 °C-42.9 °C
- Accuracy: ± 0.1 °C (between 35.0 °C and 39.0 °C);
- ± 0.2 °C (lower than 35.0 °C and higher than 39.0 °C)
- Memory: last measuring result
- Automatic power off within 10 minutes after non-use
- Fever alarm for Temp. higher than 37.8 °C
- °C/°F Switchable
- Battery: one 1.5V(LR41) button battery



Product information



Product name : Digital Thermometer(Flexible type)

Model:HK-902

measuring range:32°C~42.9°C

Accuracy: $\pm 0.1^{\circ}\text{C}$ (between 35°C and 39°C);

± 0.2 (lower than 35°C and higher than 39°C)

Memory:last measuring result

Automatic power off within 10 minutes after non-use

battery:1.5V (LR41)


°C and °F Switchable

Operating conditions: 5°C~40°C 30%-80%RH

PRODUCT CHARACTERISTICS

Fast temperature measurement

The temperature probe is sensitive and the temperature can be measured in about 60 seconds

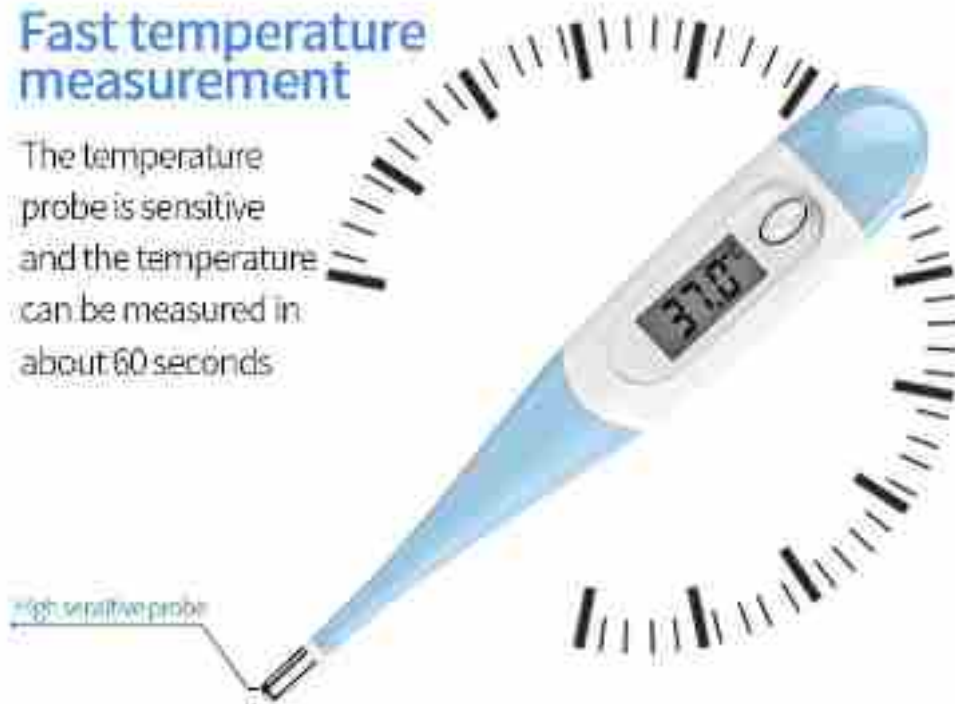

Digital thermometer

For adults and children



Three in one function

It can be used to measure oral temperature, axillary temperature and Anal temperature

- 1. Oral temperature
- 2. Axillary temperature
- 3. Anal temperature

Memory function



The last measurement value can be displayed after power on for easy query



DETAIL DISPLAY



High sensitive thermistor
Human body temperature can be measured more precisely for oral, axillary and anal

Bendable probe

The probe is bendable, convenient for oral and axillary measurement for temperature measurement



LCD display

16 LCD digital large screen, easy read the temperature value clearly, the external display easily see



Battery replaceable

Replaceable battery design, small and convenient, easy to carry



SOFT PROBE

Fahrenheit Celsius
Free switch



Automatic shutdown

When the probe is not used for 5 minutes, the thermometer will automatically shut down to save power. Press the power button to turn on the thermometer.



Beep prompt

When the probe is not used for 5 minutes, the thermometer will automatically shut down to save power. Press the power button to turn on the thermometer.



Beep prompt

When the probe is not used for 5 minutes, the thermometer will automatically shut down to save power. Press the power button to turn on the thermometer.



Battery replaceable

Replaceable battery design, small and convenient, easy to carry

USAGE METHOD

Underarm measurement method



Insert the temperature sensing part in the middle of the armpit



Counter damping armpit up

Matters needing attention

1. please keep the posture in the figure until the measurement result
2. Please wipe the sweat under the armpit before measurement. (in case of sweat, the heat conductivity changes, which may lead to inaccurate measurement)
3. Please do not touch the underwear when measuring the temperature sensing part. (when touching clothes, the heat conductivity changes, which may lead to inaccurate measurement)

Methods of oral measurement



Place the temperature sensing part under the tongue



Stay still for 60 seconds

1. The temperature will rise after eating, bathing or exercising. Please take a temperature measurement at least 30 minutes after eating, bathing or exercising.
2. Accurately grasp the daily temperature change and make records while measuring.
3. Children's temperature will rise after crying. Please take a temperature measurement when they are calm or sleeping.

PRODUCT DISPLAY



Product packaging



Size: length: 13.5cm, width: 2.5cm, height: 1.5cm

Welcome to OEM



Packing information

