

72.5mm

72.5mm

72.5mm

MASSAGE RING

Instructions



Massage Mode : 5 Modes

Power No/ Off : 5 Vibration Intensity

105mm

Product specification

Product Name: Magic Ring Product model :MRS-MR-001-BK
Charging input :5V=1A Operating voltage :7.4V
Rated power :15W Charging time: about 3.5 hours
Product material :ABS Paint process: self-repairing feel paint
Operating environment :-5 ~40

Instructions for use:

1. Long press to turn on, turn on the default gear, short press to adjust the gear, five speed cycle, five modes.
2. The charging time is about 3.5 hours. When charging, the light at the switch is steady on, and the switch light is steady on when the power is full.
3. Fully charged can be used for 5.5 hours, when the power is less than 15%, the light at the switch blinks, when the power is insufficient, the light at the switch blinks for 5 seconds and the machine will automatically shut down.
4. The machine can be used with the strap, not only to free the hands, but also to expand the massage range.

Safety function: battery with charge and discharge protection board, controller also with charge and discharge protection board, double protection is safer, longer battery life.

Product features: Massage the body through high frequency physical vibration.

Suitable for relaxing massage of many parts of the body, such as abdomen, waist, back, legs, hands, magic ring massager massage soft, massage range is larger, more flexible, more useful, compared with similar products have a higher service life.

Packing contents: Magic ring host, charging cable, manual, massager strap

MRS-MR-001-BK

Feedback & Review

Your comments and suggestions are important to us as they help us provide the best possible service. should you have any need to contact us, the info below will help you get in touch.

Kindly include your Product Model, Purchasing Date & Site, Order#, and your contact information. All your concerns will be replied.

Service Email: Abotrem@outlook.com

HOW TO GET THE WARRANTY

PLEASE LEAVE IMAGE OR VIDEO PROOFS AT THE TIME OF DELIVERY, then send the proof to Abotrem Service Email: Abotrem@outlook.com WITHIN 7 DAYS. (Note: Abotrem customer service department cannot provide any guarantee without proof)



Xiamen Morrow Sun Intelligent Wellness Equipment Co., Ltd
Address: Room A814, Tongling Square, No.3 Changle Street, Huli , Xiamen, China
E-mail: Abotrem@outlook.com